

Front of Pouch

Next Generation[®]



Universal Formula Semen Extender

Contents Produce Approximately 150 mL Liquid Volume
Not for Human and/or Animal Consumption

Guiding Your Way To Reproductive Success[™]

Manufactured For
Exodus Breeders Corporation
York, Penna 17406-9200
www.exodusbreeders.com

All ingredients are of US origin

Back of Pouch

Directions

1. **We recommend keeping our extenders at room and/or an ambient temperature of 20° C or 68° F**
2. **It is important to make sure that the semen and the extender always have like temperatures.**
3. Shake contents from the top of this pouch allowing for a clean tear between the pre-set corner notches.
4. Pour the contents of this pouch into a bottle of water
5. Make sure all contents are emptied from this pouch.
6. Replace the cap and invert slowly mixing the ingredients into solution, let stand for 20 minutes prior to use.
7. Adding semen to extender with differing temperatures induces the opportunity of shock, which dramatically reduces the optimum fertility of your stallion.

Recommended Semen Extension Protocols:

25 Million Motile Sperm per extended mL
yields an optimum environment for semen

750 Progressively Motile Sperm extended
to a minimum of 5 parts extender.

This extender has a specific pH & Osmolarity